

CEREC SW 5.2

Upgrade Description (EN)

Dentsply Sirona is pleased to announce the new CEREC SW 5.2 upgrade.

Changes made to previous version 5.1.3:

- Optimized CEREC Guide Workflow
- New patient communication possibility
 - The new “View Model” step in the model phase can be used without making any entries in the administration phase.
- New Articulator Mode - Full range dynamic occlusion
 - Functional bite registrations are calculated for a wide range of articulator settings and an overall envelope is determined.
 - This mode is "on" by default if the articulator is used and can be set via the configuration.
- Configuration
 - The time after which automatic saving should take place can be set
 - Tool wheel can be enabled/disabled
- Acquisition Phase
 - New Firmware version for CEREC Primescan
 - Export option for photos/videos
 - Recovery possibility of image catalogues after application failure
- CEREC Primemill
New Firmware version enables:
 - Fast Grinding e.max CAD for crowns
 - Extra Fine Grinding for posterior and anterior crowns, screw-retained crowns, inlays and veneers
- Further Changes
 - Export formats - Clear description texts in the export window
 - Primescan blue light scan mode starts only intraorally
 - Virtual CEREC Primemill selectable
 - Anonymization of file names for local storage
 - Light bulb to activate/deactivate feature hints
- General stability improvements and bugfixes

Introduction of two new separate applications

- DS Download Assistant
 - The DS Download Assistant guides you through the download of the CEREC SW full version files and assists you in saving the files or starting the installation.
- DS System Monitor:
 - This application enables the user or technician to quickly get an overview of the system status.
 - You can open the app by going to the hide symbols at the bottom right in MS Windows. There you open the DS System Monitor by clicking on the icon with the magnifying glass.

Installation & License

- To upgrade from a CEREC SW 5.x.x to CEREC SW 5.2 the CEREC SW 5.2 version license is required. The new CEREC Primescan firmware is included in this download and is also installed.
- CEREC SW 5.2 is a full version and does not need any preinstalled CEREC SW versions.
- DS Download Assistant can be downloaded from my.cerec.com.
- Alternatively, you can download the compressed files of the CEREC SW 5.2 from my.cerec.com.

For installation please unpack the downloaded CEREC SW 5.2 files and copy the CEREC SW 5.2 folder on your acquisition center. To start the installation, double-click the Setup.exe and follow the installation setup.

Hardware support note:

CEREC SW 5.2 supports Omnicam and Primescan scanners but requires the newer AC models with touchscreen and Windows 10 to run. Pre-2019 Omnicam units (without touchscreen) will require a Performance Upgrade to use this software. For more information on upgrading these devices, please contact your dealer representative